



Mimitakara® Rechargeable ITC Hearing Amplifier

CONTENTS


■ Charging Case

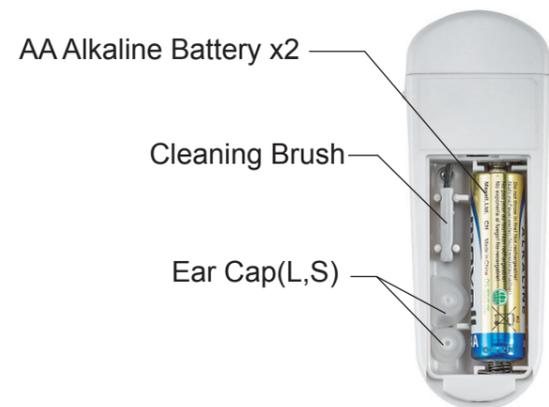

■ Hearing Amplifier

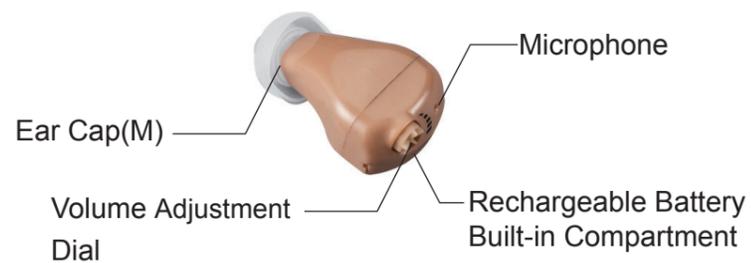

CHARGING

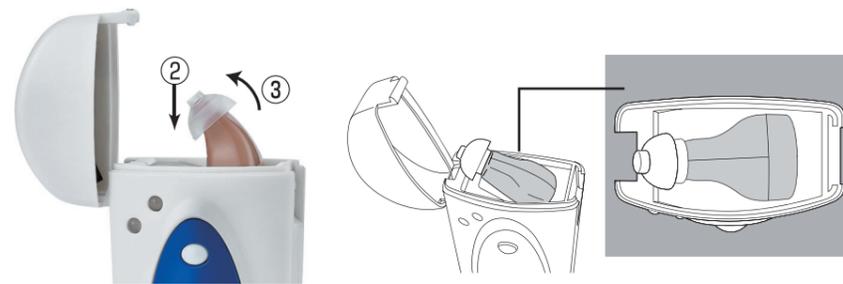

1. Insert hearing amplifier vertically as shown.
2. Lay hearing amplifier down as shown by pushing towards the charging case cover.
3. Close the lid of the charging case until it clips into place.
Please note if the hearing amplifier doesn't sit perfectly in the charging case it will not charge.

■ Charging by Battery

1. Insert a AA battery in the compartment .
2. Please check that two lamps are GREEN after you push the Status Button.
3. Please charge it for 7 hours at least .




■ Charging by AC Adapter

1. Connect the AC Adapter.
2. Please check that two lamps are GREEN after you push the Status Button.
3. Please charge it for 7 hours at least.




CHARGE STATUS

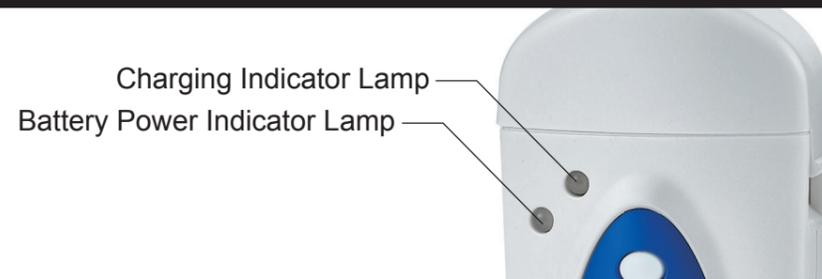

Charging Indicator Lamp	GREEN	Hearing amplifier is charging normally
	OFF	Hearing amplifier is fully charged (or hearing amplifier is not correctly inserted in the charging case)
Battery Power Indicator Lamp	GREEN	Power supply is normal
	RED	AA Battery power is low (insert new AA battery or use AC adapter)
	OFF	No power (insert new AA battery or use AC adapter)

OPERATING INSTRUCTION

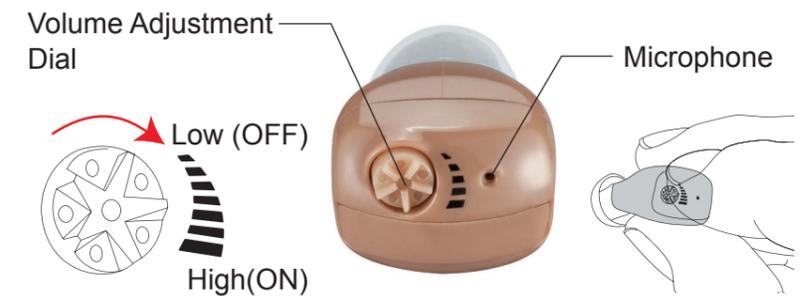

1. Turn on the hearing amplifier before putting it into your ear.
2. The power will be on when you turn the dial to the highest volume, whereas the power will be off after you turn the dial to the smallest volume.
(Please turn it off if you have finished using it.)
3. Please use your finger to grasp the microphone and block the hole.



You will hear the buzzer ringing if you put into your ear without holding the microphone.

MAINTENANCE METHOD


1. Please clean your ear before use.
If the inner ear is blocked with ear wax, then sound quality will be reduced.

Before each use



2. Please clean out any earwax in the hearing amplifier.
Earwax can most often congregate around the sound tip as show below.

After each use

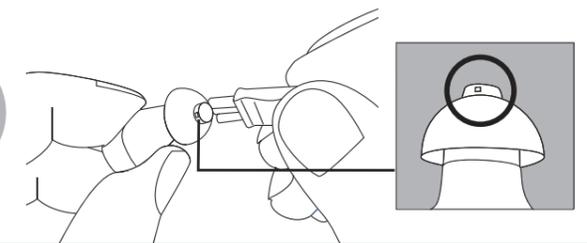

TROUBLE SHOOTING

1. Is power on?

Turn on the power switch.

2. Is the ear cap blocked with wax or debris?

Clean the ear cap using the cleaning brush provided.

3. Is the battery dead?

Place the hearing amplifier in the charging case, and ensure it is charging by battery or AC adapter.

Persons suffering from loss of hearing should have a medical evaluation for diseases of the ear periodically after using hearing amplifiers.

SPECIFICATION

General Title of Commodity : Rechargeable ITC Hearing Amplifier

Purpose & Function : For assisting people of hearing obstacle

Standard Frequency : 1600Hz

Max. Output Sound Level Gain : 35±5dB

Highest Sound Pressure Level for Input SPL 90dB : 118dB±5dB

Equivalent Input Noise Level : Below 29dB

Built-in Rechargeable Battery Current : < Below 0.8mA/2.5V

Rechargeable Time : Up to 500 times in total (it may be different because of depending on what level of volume you use.)

Continuous Use : 7~12hours /each recharged time.

Operating Temperature / Humidity : +10°C ~ +40°C, 30% ~ 85% RH

Storage Temperature / Humidity : -10°C ~ +60°C, 30% ~ 85% RH

Size : <Hearing Amplifier> 10.5mm(L)x14.4(W)x24.0mm(H)

<Rechargeable Case> 39.0(L)x 22.0(W)x103.0mm(H)

Weight : <Hearing Amplifier> 2.4g / <Charging Case> 28g

Material : <Ear Cap> silicone

<Hearing Amplifier > / <Charging Case> / <AC Adapter> ABS



Very Important Notice ~ Recharge Any Time~

Under normal conditions, the battery can be recharged and discharged approximately 500 times.

In order to extend the rechargeable battery life, we strongly recommend users to charge the hearing amplifier whenever it is not in use.

1. Please Note: There is no need to recharge your hearing amplifier until it is almost out of power. To charge your hearing amplifier whenever you are not using it will be helpful to maintain the rechargeable battery & extend the usage life for recharging & discharging **up to 500 times.**
2. According to the character of rechargeable battery, when it is almost run out of power, the sound from hearing amplifier will reduce. Please don't treat it as any defective, just put your hearing amplifier back to the charging case or use the AC adapter to recharge your hearing amplifier.

NOTICE

- To extend the rechargeable battery life, fully charge the hearing amplifier and then switch off when not in use for a long period of time.
- Keep both your ears and the hearing amplifier, free from earwax.
- With proper care, the rechargeable battery can last for up to, and beyond, 500 charge cycles.
- When the rechargeable battery is low, the sound becomes quieter. This means the battery is low, and needs to be charged.
- Keep this product away from infants, children and pets.
- Prior to first use, read instructions carefully to ensure proper operation, care and handling of your hearing amplifier.
- Do NOT use this product under the following conditions : operating mobile phone, sleeping, showering, washing your hair / face, swimming, or any vigorous exercise, to avoid damage to the hearing amplifier or you.

SITUATION USE

1. Conversations :

Keeping the average volume is fine.




2. Watch TV :

Let's enjoy your favorite program!
You won't interfere with family at all.



3. Bird-watch :

Clearly hear "Chirrup" in the distance.

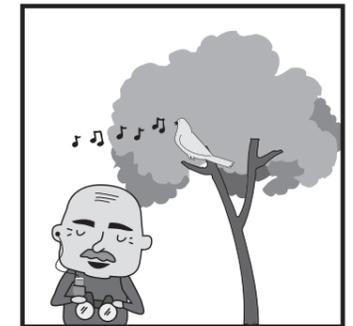

4. Listening to a speech :

Even sitting at the back, you will have no trouble hearing a speech.

