

# Mimitakara<sup>®</sup> JAPAN


## BTE B/O Hearing Amplifier

### Check the following components :

- Hearing Amplifier x 1
- Ear Buds x 2 ( L and S size x 1pc)  
(S size is already attached to the hearing amplifier)
- Zinc Air battery P13 x 1
- Portable Case x 1



### Product Illustration :

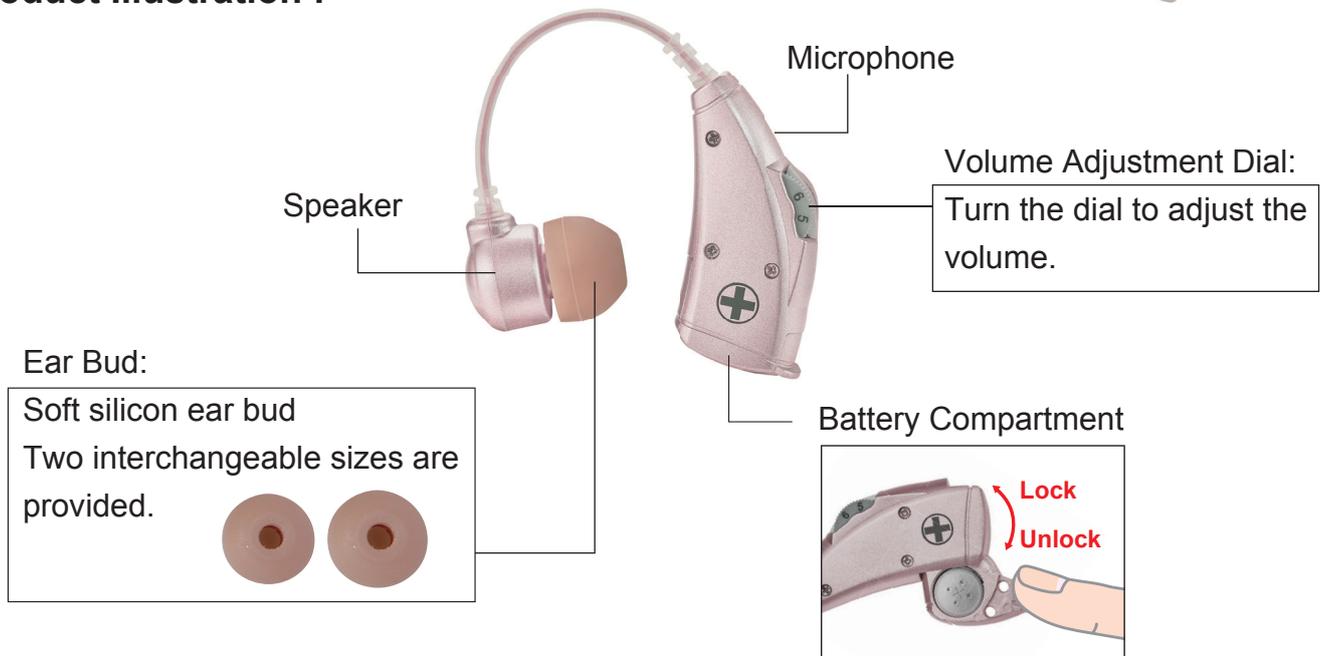

### Insert and Replacing the Batteries :

1. Open the battery compartment gently and insert the battery. Only use the standard recommended the Zinc Air battery P13. (Remove the old one if this is not the first time using.)
2. Place the new battery in the compartment with positive (+) terminal facing outwards.
3. Close the battery compartment. The hearing amplifier will automatically be powered on.

### Note:

- To avoid damaging the battery compartment, DO NOT forcibly remove the battery or open / close the compartment.
- If adjusting the volume to its highest setting, the sound is barely heard or there is no sound at all. This indicates that the battery needs to be replaced.

## To Turn Power On & Off?



Insert a battery and completely close the battery compartment.



Open the battery compartment until the battery is no longer touching the battery contacts.

## How to Use the Hearing Amplifier?

1. The hearing amplifier comes with 2 different ear bud sizes (large & small). Please select the size that best fits your ear.
2. Place the hearing amplifier into your ear with the amplifier's hook behind your ear.
3. Adjust the volume to a suitable level. If you hear a harsh sound, it means the volume setting is too high or the hearing amplifier is not placed in your ear properly. Adjust the position of the hearing amplifier or reduce the volume setting slightly to resolve this problem.
4. Take the hearing amplifier out of your ear when you finish using it and place it in the portable case. Ensure the hearing amplifier is in the OFF mode (Take out the battery) before placing it in the case.



## Specifications:

- Battery type: Zinc Air battery P13
- Continuous Use: Up to 95 hours
- Max. Output Sound Level Gain: 29+/-5dB
- Size: 35.4(L) x 15.6(W) x 8.7(H) mm / Weight: 8g

## Important Points

1. If you experience any discomfort while using your hearing amplifier, please consult with your doctor.
2. Place the hearing amplifier gently in your ear so that you feel comfortable. DO NOT force it into your ear.
3. In order to safeguard personal hygiene, please do not share your amplifier with anyone else.
4. The hearing amplifier contains small parts. Keep out of reach of children.
5. When playing intense sports or exercising, please do not wear your hearing amplifier.
6. Please do not wear your hearing amplifier when washing your face, when it is raining or when there is the possibility of it becoming wet. Also, do not wear it when sleeping.
7. Take out your hearing amplifier when using a mobile phone.
8. Open battery compartment (into "off" mode) your hearing amplifier when not in use and place it in the portable case and close the cover. This will prevent unnecessary battery usage and preserve the battery's life.
9. If you wear any other electronic instrument, such as a pacemaker, etc., you MUST first get approval from a doctor before using the hearing amplifier.
10. Store the hearing amplifier in a cool place, away from direct sunlight and places of high humidity.